GR EN

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T: +30 28210 63222 | WEB: www.mobiakcare.com

PRODUCT DESCRIPTION – INTENDED USE

The current device are intended to lift and move people with motor difficulties. The lift is suitable both for hospital and home use.


CAUTION: The lift supports max load up to 200 Kg. Don't get over this limit. The lift has breaks in the rear wheels.

ASSEMBLY


Unpack all the parts carefully and lay them down.


- Remove the nut covers from bolts and nuts so that you can use the hexagonal wrench provided.
- Connect the support column to the base by securing it with bolts "A" and "B" highlighted below and tightening them using the wrench provided.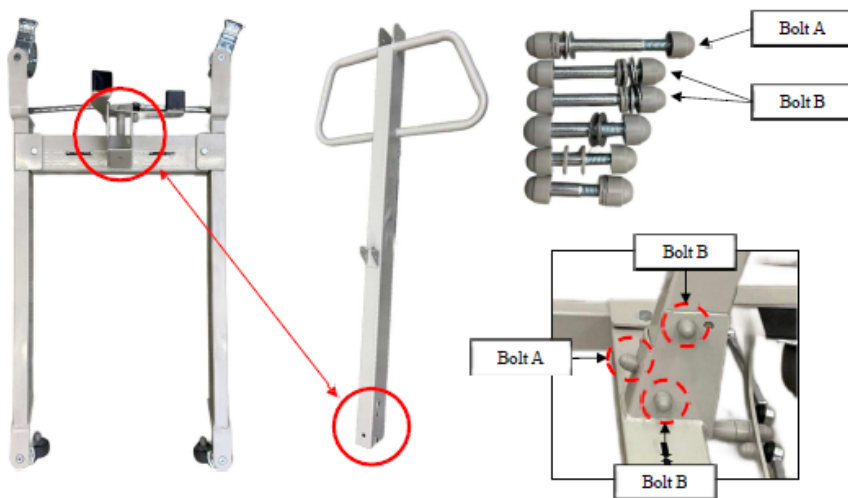

- Connect the lifting arm to the support column by bolt "C".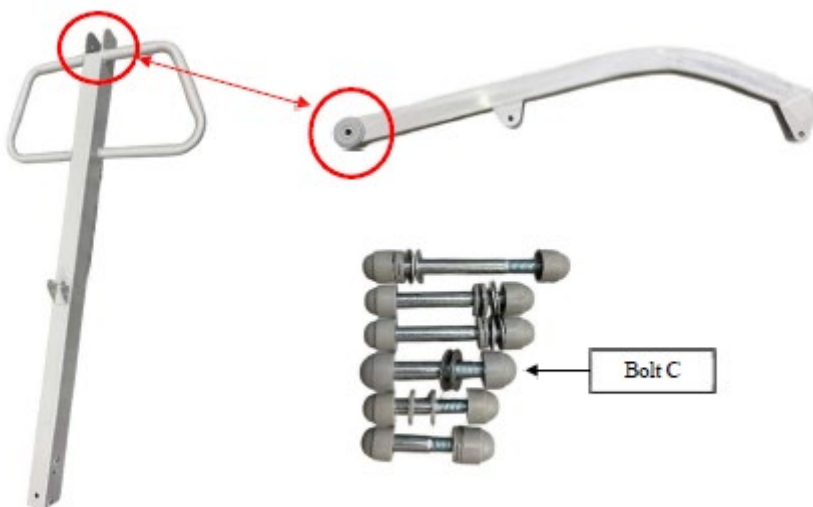


- Connect the ends of the actuator to the joint located on the support column and support arm, using bolts "D".

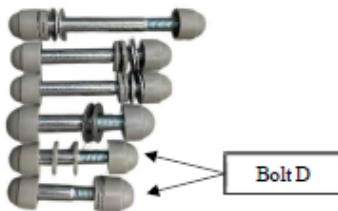- Remove the bolt present on the lifting arm with hooks.
- Connect the lifting arm with hooks to the end of the lifting arm base and tighten the bolt with the wrench given.

- Pull out the battery (see next page) so that the No. 2 center slots of the battery frame (blue circles fig.1) are visible.
- Place the back of the battery frame (fig.3) on the support bracket attached to the main column (fig.2) and slide it down so that the No. 4 metal protrusions fit into the No. 4 slots in the frame (red circles).
- To prevent the battery frame from dislodging from the main column, screw the No. 2 screws given in the package into the slots (blue circles).

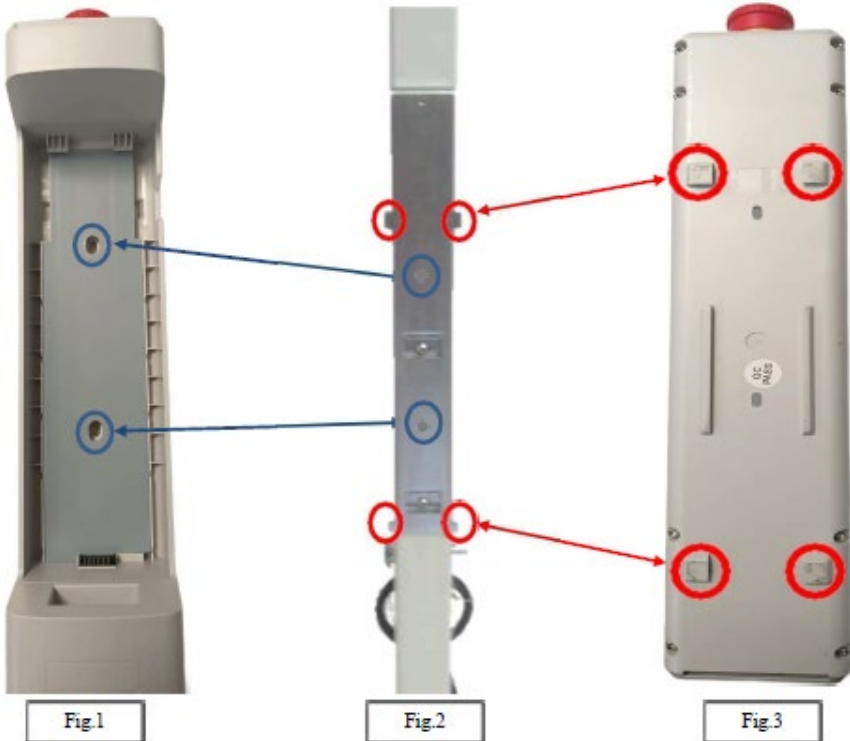- Connect the hand control cable to the control unit by inserting the relevant 5-pin connector, into the appropriate slot (HC), and connect the actuator to the control unit by inserting the relevant 4-pin connector into the appropriate slot (M1).





- To pull out the battery, act on the lever at the top and lift the battery as described in the figure.

TECHNICAL CHARACTERISTICS

Input Voltage	220V – 50Hz
Product weight (Kg)	45
Max load(Kg)	200
Maximum lifting arm height (upper end) [A] (cm)	200
Minimum and maximum lifting height (distance of lifting arm hooks from the ground) [B] (cm)	90 - 172
Height of the bottom surface of the base from the ground [C] (cm)	4.5
Height of the top surface of the base from the ground [D] (cm)	11
Minimum depth [E] (cm)	125
Internal length of base [F] (cm)	80
Minimum inside width of closed base [G] (cm)	49
Minimum outer width of closed base [H] (cm)	60
Bend radius range [I] (cm)	131
Internal width of open base [L] (cm)	80
External width of open base [M] (cm)	91
Diameter of front wheels (cm)	7.5
Diameter of rear wheels with brakes (cm)	10

TECHNICAL CHARACTERISTICS

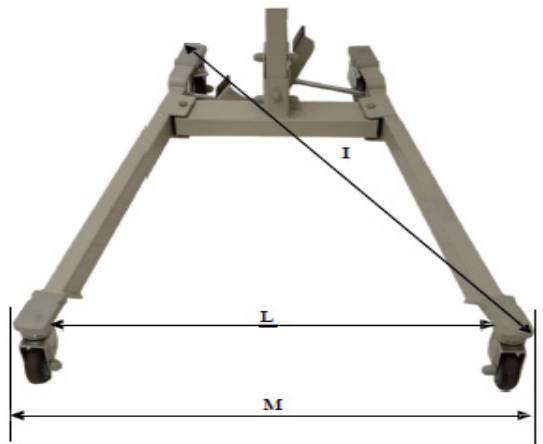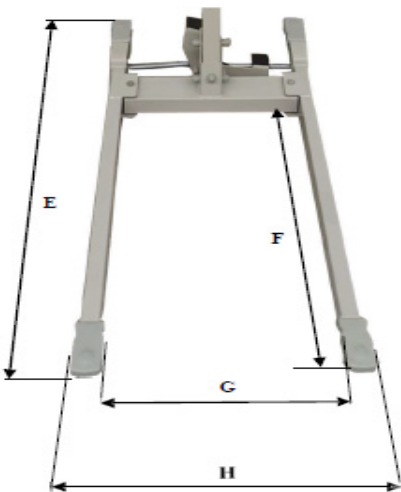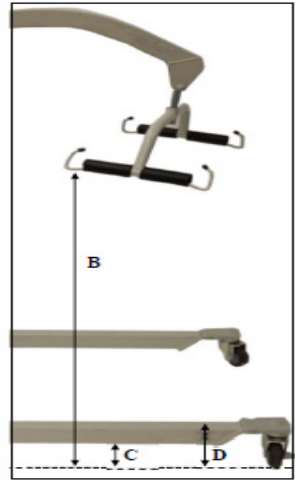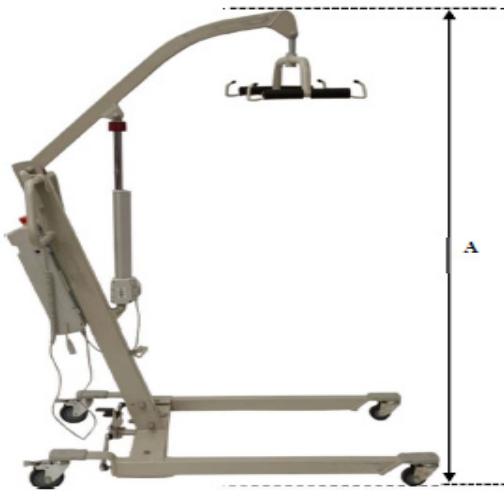INCIDENT REPORTING:

Any serious incident that occurs in relation to use of the device must be reported to the manufacturer and a competent authority of the Member State in which the user and / or patient resides

PRECAUTIONS

- Read and understand these instructions before assembling or using this product. If you do not understand some of these instructions, please contact with a healthcare professional for instructions on how to use this product.
- If the lift is not correctly assembling it could be caused injury
- Check periodically the screws and you have to tight these if it is necessary.
- If parts are damaged or missing, contact with your dealer immediately. **DO NOT** use other spare parts. Using non-genuine spare parts could cause injuries or damage to the lift.
- Do not expose the product to temperatures above 40 °C for a long time, as this may cause damage to the product and consequently possible injury to the user.
- MOBIAK SA does not take any responsibility for any damage or injury caused by improper assembly or misuse of this product.
- Make sure that the product has been assembled correctly as the instructions refer and all parts are securely fastened, before use

CLEANING AND DISINFECTION

Use only a damp cloth and mild soap on surfaces where they need cleaning. Then dry thoroughly before using the product.

If you have to disinfect the product, use a common, mild disinfectant.

Never use acids, alkalis or solvents such as acetone or dissolving fluids.

IMPORTANT (!)

In case of damage or malfunction of your product please contact the authorized and trained representative

In the event of a power failure, lower the Bordeaux safety round and manually turn the lifting tube.

In case of emergency, press the red safety button.

ATTENTION (!)

Use of the following product is prohibited for purposes other than those specified in this manual

MOBIAK SA disclaims any responsibility for damage resulting from improper use of the product or use other than that set forth in this manual.

MOBIAK SA reserves the right to make changes to the product and the following manual without notice in order to improve its features.

DECLARATION OF CONFORMITY

We are solely responsible for declaring that the Medical Devices mentioned in this statement are of Low-Risk Class (Class I) and comply with the requirements of the European Regulation 2017/745 and where appropriate, the standards and legislation referred to.

ENVIRONMENTAL PROTECTION

If one day you find out that your product needs replacement or it is no longer useful for you, consider protecting the environment:

- 1) Do not dispose of your product with other municipal waste (this is also the meaning of the recycling symbol listed).
- 2) Contact your local authority to tell you where your appliance is to be recycled.
- 3) Having your junk device in the right place recycling helps protect the environment as well as the material handling of your product
- 4) The electrical products due to their construction materials, if not correctly disposed, may lead to environmental and furthermore health hazards.

WARRANTY

The product is guaranteed for a period of two* (2) years from the purchasing date.

The warranty covers factory defects & **DOES NOT** cover damages resulting from misuse, poor maintenance, modification, overuse or non-compliance with the product's operating instructions. Also **DOES NOT** cover the parts that can wear out during use or over time.

Damages or defects caused by: natural disasters, unauthorized maintenance or repair, supply problems (where applicable), carriers are not covered by the warranty.

* **The battery has six (6) months warranty.**

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Description of symbols that have been printed on the label, Master Carton and or User Manual

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MOBIAK S.A