**REF ELITE-TIM**

- PL INSTRUKCJA OBSŁUGI**
  - CS NÁVOD K OBSLUZE**
  - DA BRUGSANVISNING**
  - DE BENUTZERHANDBUCH**
  - EL ΟΔΗΓΙΕΣ ΧΡΗΣΗΣ**
  - EN USER MANUAL**
  - ET KASUTUSJUHEND**
  - HR KORSNIČKE UPUTE**
  - HU FELHASZNÁLÓI ÚTMUTATÓ**
  - IT MANUALE D'USO**
  - LT NAUDOJIMO INSTRUKCIJA**
  - LV LIETOTĀJA INSTRUKCIJA**
  - PT MANUAL DO UTILIZADOR**
  - RO MANUAL DE INSTRUCȚIUNI**
  - SK NÁVOD NA POUŽITIE**
  - SL NAVODILA ZA UPORABO**
  - SV BRUKSANVISNING**
- 
- A large, faded image of a wheelchair is positioned in the background, partially overlapping the list of languages. The wheelchair is shown from a side-rear perspective, highlighting the seat, backrest, and wheels.

<b>PL</b> INSTRUKCJA OBSŁUGI .....	5
<b>CS</b> NÁVOD K OBSLUZE .....	11
<b>DA</b> BRUGSANVISNING .....	16
<b>DE</b> BENUTZERHANDBUCH .....	21
<b>EL</b> ΟΔΗΓΙΕΣ ΧΡΗΣΗΣ.....	27
<b>EN</b> USER MANUAL .....	33
<b>ET</b> KASUTUSJUHEND .....	38
<b>HR</b> KORSNIČKE UPUTE.....	43
<b>HU</b> FELHASZNÁLÓI ÚTMUTATÓ .....	48
<b>IT</b> MANUALE D'USO.....	53
<b>LT</b> NAUDOJIMO INSTRUKCIJA .....	58
<b>LV</b> LIETOTĀJA INSTRUKCIJA .....	63
<b>PT</b> MANUAL DO UTILIZADOR .....	68
<b>RO</b> MANUAL DE INSTRUCȚIUNI.....	73
<b>SL</b> NAVODILA ZA UPORABO.....	78
<b>SV</b> BRUKSANVISNING .....	83

## 1. Wykonanie

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## ELEMENTS OF THE WHEELCHAIR

- |                               |  |
|-------------------------------|--|
| 1. Handgrip                   | 10. Footplate                          |
| 2. Attendant brake            | 11. Footrest lever                     |
| 3. Backrest folding mechanism | 12. Seat with security belt            |
| 4. Armrest                    | 13. Backrest                           |
| 5. Armrest adjustment clip    | 14. Anti-tip wheels                    |
| 6. Parking brake (hand brake) | 15. Centre of gravity adjustment       |
| 7. 8" swivel, front wheel     | 16. 24" solid, rear wheel with handrim |
| 8. Quick-release mechanism    | 17. Pocket for goods                   |
| 9. Calves protective strap    | 18. Cane holder                        |

## **1. Characteristics**

The active wheelchair is made of a lightweight aluminum frame. It features flip-up and adjustable armrests. It is equipped with a quick release function, centre of gravity adjustment (front and back), attendant brakes and a foldable backrest. It comes with a protective straps for calves and security belt, anti-tip wheels with length adjustment, solid 24" rear wheels and 8" front swivel wheels. Additionally, breathable mesh upholstery allows the user to fold the wheelchair without detaching it from the frame.

**The set includes:** wheelchair frame with wheels, upholstery with pocket for goods, footrests with protective straps, anti-tip wheels, cane holder, set of keys for assembly/dismantling.

## **2. Application**

The active wheelchair is designed for people who have problems with unassisted moving and for patients with dysfunctions and reduced mobility. This self-propelled wheelchair can also be pushed by an accompanying person. The wheelchair is suitable for indoor and outdoor use.

## **3. Assembly**

### **Step 1. Unfolding the wheelchair**

Place the wheelchair on the ground. Unfold the wheelchair using the armrests. Then press the sides of the seat so that all the elements fit together.

### **Step 2. Backrest and seat**

Lift the backrest upwards and lock it with backrest locks which are located below the hand grips (3). Fix the upholstery with bag on the wheelchair.

### **Step 3. Footrests**

Fix the footrest on the outer side of the frame. Then turn the footrest inwards (press the footrest lever at the same time) until it is locked.

### **Step 4. Armrests**

The armrests are tiltable. To raise the armrest, press the button located on the frame (5) of the wheelchair.

### **Step 5. Height adjustment of the footrests and armrests**

- Adjust the footrests using set of assembly keys. To adjust, unscrew the screw located on the footrest frame. Enough space should be left between the footplates and the ground as footplates set too low may cause difficulties in climbing hills and curbs. Adjust the footplates so that the angle between trunk and thighs is  $90^{\circ}$  (optimum sitting position for the user).
- To adjust armrests pull the adjusting clamp and pick the desired height. Both armrests should be on same height level.

## **Step 6. Assembly of protective straps and cane holder**

Attach the protective straps to the footrests to prevent the user's legs from sliding down the footplates. Mount cane holder on the preferred side of the wheelchair.

### **FOLDING**

**The wheelchair can be re-folded in order to be transported and stored more easily:** Swing away the footplates and set them in vertical position or remove. Detach the seat and backrest upholstery with bag. Lift the seat upwards by grabbing it from back and front until the frame of the wheelchair is folded.

### **BACKREST FOLDING FUNCTION**

- To fold the backrest, simultaneously press locking elements which are located under the hand grips at the back of the backrest and pull the backrest downwards.

### **PARKING BRAKE**

To lock the wheels, push the lever forward until the wheel is fully blocked (separately for left and right wheel). Please, note that the parking brake only serves to lock the wheels when the wheelchair is at a complete stop.

### **ATTENDANT BRAKES**

The levers of the brakes are located behind the backrest, under both grips.

- In order to stop the wheelchair, squeeze the lever of the brake handle. To release the brake, let go of the levers. Perform this activity smoothly and with appropriate force.
- The brakes are equipped with parking locks. In order to use them, squeeze the top lever of the brake handle till you hear "click" sound. To release parking brakes, squeeze the bottom lever of the brake handle.

### **PARKING BRAKE ADJUSTMENT**

The correct lock of the parking brake depends on its adjustment and position. The handle of the wheel lock should be positioned so that the brake shoe blocks the wheel firmly and the lever does not face excessive resistance.

- In case of excessive resistance, pull the handle slightly forward by twisting off the locking screws which are located on the parking brake. Set the brake shoe in such position that it blocks the wheel easily without excessive resistance.
- When the parking brake shoe does not touch or lock the wheel sufficiently, follow the above instruction by moving the handle in the opposite direction.

## **4. Before use**

### **Step 1. Checking the condition of the wheelchair**

Check the condition of the brakes, construction and tires. Check if all the components are assembled and adjusted properly.

**Note!** Do not use the wheelchair if any defects are noticed.

### **Step 2. Using parking brakes**

Lock the wheels when getting in or out of the wheelchair.

### **Step 3. Footplates**

The footplates should be set in vertical position when getting in or out of the wheelchair. Do not use the footplates as a support while getting in or out of the wheelchair, due to the risk of falling. After transferring to the wheelchair, unfold the footplates.

**Note!** Do not stand on the footplates due to the risk of damage.

### **Step 4. Getting in the wheelchair**

The wheelchair should be located as close to the patient as possible. The wheelchair should touch the bed or other seat being used. If the user cannot change their seat independently, transfer the user to the wheelchair.

### **Step 5. Position of the user**

The user should sit in an upright position and their knees should be bent at the angle of 90°.

**The wheelchair is ready to use.**

## **5. Technical data**

<b>Seat dimensions:</b> 44/46/49 x 43 cm	<b>Seat height from the ground:</b> 49 cm
<b>Overall width:</b> 61/63/66 cm	<b>Armrest height from the seat:</b> 23-29 cm
<b>Overall depth:</b> 110cm	<b>Folded dimensions:</b> 83 x 26 x 77 cm
<b>Overall height:</b> 95 cm	<b>Weight:</b> 11,60-12,00 kg
<b>Maximum capacity:</b> 120 kg	

## **6. Notes**

**Timago International Group does not bear responsibility of improper usage of the product, failure to comply safety regulations and misuse.**

## **7. Cleaning and maintenance**

Carry out the maintenance of the wheelchair components regularly:

- The upholstery and the cushion can be cleaned with damp (not wet) cloth and cold or warm water, without the use of detergents. Difficult stains can be cleaned with a sponge. Do not soak the upholstery too much. Dry away from heat sources.
- Plastic elements should be cleaned with damp cloth possibly with the use of appropriate detergents for plastic elements. In such case follow the safety measures and instructions given by the manufacturer.

## 8. Storage and transport conditions

The product should not be exposed to direct sunlight, water or moisture.

## 9. Warranty information

All products distributed by our company are covered by a warranty, the terms of which are described in the warranty card available on our website. Please contact the retailer from whom you purchased the product. Please, keep in mind that for warranty purposes, the proof of purchase (receipt or invoice) should be retained.

## 10. Medical device incidents

Any serious incident involving a medical device must be reported to the manufacturer and to the competent authority of the European Union member state where the user or patient resides.

## 11. Labels

	Reference number		Manufacturer
	LOT number		UDI code
	Serial number		Medical device
	Note		Please read the instruction
	The product meets the legal requirements of the European Union for medical devices (MDR 2017/745)		

Our complete line of products is available from our distributors. Learn more at: [www.timago.com](http://www.timago.com). Thank you for choosing Timago!

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