REF EASY-TRANSFER

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BRUGSANVISNING

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Product code: EASY-TRANSFER

1. Characteristics

Transfer board is made of durable plastic with handle for easier transportation and storage. Thanks to its curved shape and low-friction surface it allows for easy and safe transfer of patient, without excessive physical effort or strain on the caregiver's body.

2. Application

The transfer board is designed for transferring patients in medical facilities and home environments. It can be used to assist caregiver in moving patients on wheelchair, chair or armchair. The use of the transfer board has a positive effect on caregiver safety, effectively reducing the risk of back and muscle strain, and enhancing patient comfort thanks to reduced friction, which helps prevent bedsores.

3. Technical data

Board size	80 x 21 x 1,2 cm
Material	ABS

4. Notes

Timago International Group does not bear responsibility of improper use of the product, failure to comply safety regulations and misuse.

5. Maintenance

Transfer board can be cleaned with use of soap and damp cloth. Dry thoroughly with clean cloth. Difficult stains can be cleaned with sponge.
Note! Do not expose product to sun for longer period of time. It may result in board discoloration.

6. Storage and transport conditions

Product should not be exposed to direct sunlight, water or moisture.

7. Warranty information

All products distributed by our company are covered by a warranty, the terms of which are described in the warranty card available on our website. Please contact

the retailer from whom you purchased the product. Please keep in mind that for warranty purposes the proof of purchase (receipt or invoice) should be retained.

8. Medical device incidents

Any serious incident involving a medical device must be reported to the manufacturer and to the competent authority of the European Union member state where the user or patient resides.

9. Labels

	Reference number		Manufacturer
	LOT number		UDI code
	Serial number		Medical device
	Note		Please read the instruction
	The product meets the legal requirements of the European Union for medical devices (MDR 2017/745).		

Our complete line of products is available from our distributors. Learn more at: www.timago.com. Thank you for choosing Timago!

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