

INSTRUCTIONS FOR USE

IV stand with wheels , foldable, 4 hangers

Product code: STAND-TIM



1. Characteristics

The IV stand is made of aluminum, it has 4 durable hangers and plastic base with five wheels with locks. Owing to simple height adjustment, it is possible to adjust the stand to the user's individual needs. The stand has a folding function which makes it easy to transport the product.

2. Application


The IV stand with wheels is ancillary equipment which facilitates administration of infusion liquid. The IV stand is designed to be used at home or in hospital.

3. Assembly and adjustment

Assembly

Step 1. Insert lower tubing (D) into the mounting hole located in the base with wheels (E). It is a tube marked with the number "2" with a metal sleeve at the top.

Step 2. From the bottom of the base (E), fix the spacer (F), the spring washer (G) and tight the screw(H).

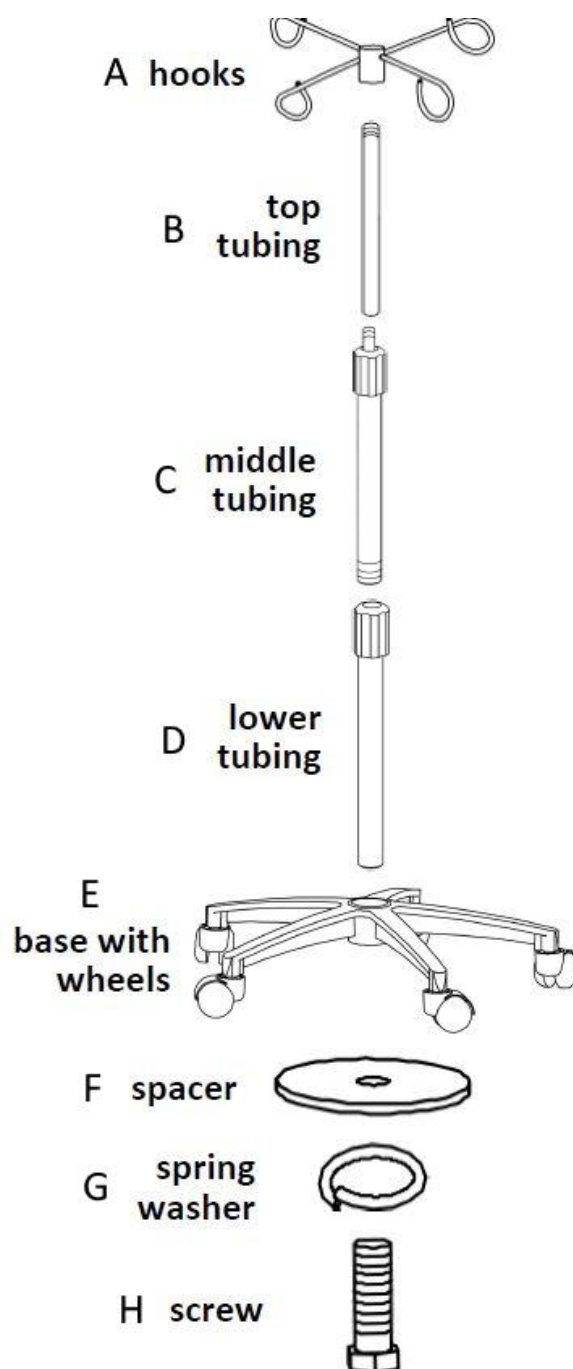

Step 3. Inside the lower tubing (D) is placed a tube marked with the number "1", which should be left inside for further assembly.

From the second tube marked with the number "1", remove the black securing cap (it will not be used in further assembly). Then screw together the two tubes marked with the number "1".

After connecting two tubes together, the top tubing (B) will be ready.

Step 4. Overlap the middle tubing (C) on top of the previously prepared top tubing (B) and screw it into the lower tubing (D).

The middle tubing (C) is a tube marked with the number "2" with a plastic sleeve in the upper part.

Step 5. Lift the top tubing (B) slightly upward and screw the hooks (A) onto it.

Tighten the plastic sleeve, locking the height of the stand.

The IV stand is ready for use.

Height adjustment

The plastic sleeve in the center of the IV stand is used to adjust the height of the entire IV stand. To adjust, loosen the plastic sleeve, set the correct height, then tighten the plastic sleeve locking the new setting.

Brakes

The IV stand has 5 wheels with brakes, which allow you to lock the IV stand in the selected location.

4. Technical data

Height adjustment	119 -218 cm
Weight	2,54 kg
Max. capacity of one hanger	2,5 kg

Base diameter	56 cm
Base height	13 cm
Wheels	5x 2'' in diameter

5. Notes

Timago International Group does not bear responsibility of improper use of the IV stand, failure to comply safety regulations and misuse.

6. Maintenance

Clean the IV stand using water, sponge and mild disinfectants, then dry thoroughly. Do not use for cleaning: strong detergents, abrasives, bleach, gasoline, solvents.

Check systematically: tightening of mounting elements, condition of hangers with hooks, condition of wheels with brakes

7. Storage and transport conditions

Device should not be exposed to direct sunlight, water or moisture.

8. Warranty information

All products distributed by our company are covered by a warranty, the terms of which are described in the warranty card available on our website. Please contact the retailer from whom you purchased the product. Please, keep in mind that for warranty purposes, the proof of purchase (receipt or invoice) should be retained.

9. Labels

	Reference number		Manufacturer
	LOT number		Manufacture date

	Serial number		Medical device
	Note		Please read the instruction
	The manufacturer checked for compliance with essential medical devices requirements.		

Our complete line of products is available from our distributors.
Learn more at: www.timago.com. Thank you for choosing Timago!



Timago International Group
Spółka z o.o. i Spółka – Spółka komandytowa
ul. Karpacka 24/12
43-316 Bielsko-Biała, Polska

T.: +48 33 499 50 00
F.: +48 33 499 50 11
E.: info@timago.com



09/2024(II)